
# Ileo Gel+

---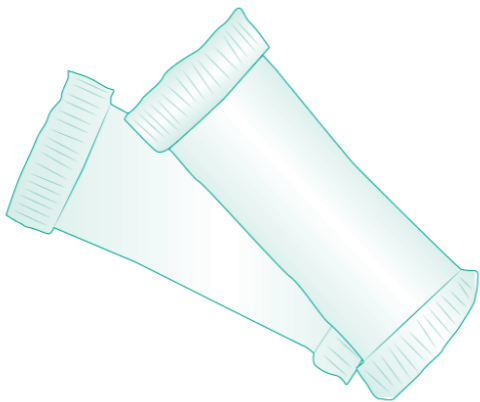

---

**B | BRAUN**

---

## EN Instruction for use

### What Ileo Gel+ is and what is used for:

B. Braun Ileo Gel+ is a gelifying agent to be used in conjunction with ostomy collecting bags.

### Indication:

Thickening of liquid stools within ostomy pouches, recommended for enterostomates.

### Clinical benefits & performances:

Ileo Gel+ transforms liquid stools into a gel and makes them easier to manage, in particular at pouch emptying. It also reduces the risk of the filter getting clogged, thus preserving its efficacy for a longer time.

### Composition:

Sodium polyacrylate powder (hydrosoluble sachet).

**Patient population:** ileostomates and colostomates with liquid stools, adults or children above four years.

**Contraindication/residual risks and side effects:** there are no formal contraindications, residual risks or side effects for the use of Ileo Gel+.

### Using the product:

1. Dry your hands thoroughly before use.
2. Insert the sachet into the pouch.

#### ■ For 2-piece systems:

- Before setting up the pouch, insert the sachet through the upper pouch opening.
- After emptying the pouch, a new sachet can be introduced into the pouch via the outlet of the pouch. Be careful not to squeeze the sachet.

#### ■ For 1-piece systems:

- Before setting up the pouch, insert the sachet through the upper pouch opening.

- 
- After emptying the pouch, a new sachet can be introduced into the pouch via the outlet of the pouch without putting pressure onto the sachet.
3. When the pouch is full, empty it through the outlet into the toilet only.

### Important!

- If you experience any difficulties when inserting the sachet, moisten the sachet with water for 3 to 5 seconds maximum. The sachet will become slippery making it easier to insert.
- Do not keep the wet sachet in your hands.
- One sachet will normally be enough to gel the pouch contents. However, depending on the volume of stools, a second sachet may be introduced.
- Do not insert more than 2 sachets simultaneously - this may solidify the stools too much and hinder easy pouch emptying.

### PRECAUTIONS FOR USE:

- DO NOT INGEST
- INSERT THE SACHET(S) WHILE STANDING OR SITTING
- DO NOT LAY DOWN UNTIL THE FIRST EFFLUENTS ARE SOLIDIFIED
- KEEP AWAY FROM THE REACH OF CHILDREN
- CONTAINS SULFITES
- KEEP AWAY FROM HUMIDITY
- RESPECT THE STORAGE TEMPERATURE PRINTED ON THE BOX AFTER REMOVAL OF THE PROTECTIVE PLASTIC FILM
- DO NOT USE AFTER THE EXPIRY DATE

National regulations for the destruction of potentially infectious materials must be observed.

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**JP** B. Braun Aesculap Japan CO., LTD
**SE** B. Braun Medical AB
**SV** DISMED, S.A. DE C.V.
B. BRAUN MEDICAL SAS