



URINOCOL® PEADIATRIC URINE BAGS FOR BOYS AND GIRLS Looking for adapted solutions

Urinocol® Premature

For baby from 1200 g to 2500 g Bags for premature infants

• 2 sizes: Mini and Standard depending on baby weight.

« Parachute » shape to maintain the bag more effectively in place.



With extension line: connection to urine bag adapted for incubator.

Urinocol® Paediatric



For baby > 2 500 g

- High skin tolerance adhesive: gentle on application and removal.
- Adhesive shape adapted to boy and girl anatomy.
- Transparent: permanent visual control of urine.
- Material: PVC free pouch / Latex free adhesive.

Anatomically shaped adhesive



Adaptation for girl and boy anatomy.

NO DRAINAGE

Closed bag

• Urine sampling for biochemical, cytological or bacteriological examination.

With DRAINAGE

Opened bag

• Collecting bag for diuresis management.

Urinocol® Paediatric

Application step by step for girls:

Choose the right size! Make sure that the size of the pouch is adapted to the weight of the child.
Skin preparation: dry, free of oily components.
Follow the application procedure.



Remove and discard central panel of the adhesive. Remove the inferior part of the release paper.



Apply with gentle pressure beneath the baby's penis or vulva.



Remove the superior part of the release paper.



Apply with gentle pressure around the baby's penis or vulva.

Urinocol[®] Paediatric Premature

Urinocol Paediatric ordering information

			Reference	Units/ box
	Sterile open paediatric bags	Воу	227546	50
		Girl	227566	50
67 Qa	Sterile closed paediatric bags	Boy	227550	100
		Girl	227560	100

Urinocol Premature ordering information

		Weight	Reference	Units/ box
Sterile St premature	andard	1200- 2500 g	227571	50
bags	Mini	1200 g	227570	50
Sterile premature bags (with	andard	1200- 2500 g	227573	50
extension line*)	Mini	1200 g	227572	50

*made of PVC: DEHP free

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun.

Non-binding documents and photographs.

B. Braun Medical declines all liability for the use of this support and particularly for damage suffered y the user resulting from the use of this support or the impossibility of using it

Commercialized and manufactured by:

B. Braun Medical SAS | 204 avenue du Maréchal Juin | 92100 Boulogne Billancourt – France Tel. 01 41 10 53 00 | Fax 01 41 10 53 99 | www.bbraun.com/continence-care-and-urology B. Braun Medical SAS with a share capital of 31 000 000 € | RCS Nanterre 562050856