

Ecoflac[®] Connect S, M, and L

Handling instructions for antiinfective admixture

3 DEVICES, 1 HANDLING PROCESS Ecoflac[®] Connect M used here, demonstrates the handling process.

1 Insert Ecoflac[®] Connect



Remove either one of the tamper-evidence sealing foils of the Ecoflac[®] plus. Spike transparent end of Ecoflac[®] Connect into the germ-free port, which does not require disinfection prior to use.

2 Connect the vial



After disinfection of the rubber surface of the vial, insert into the opaque end of the Ecoflac[®] Connect. A "click" sound confirms the secure connection.


- Ecoflac® Connect S fits vials of 13 mm rubber diameter
- Ecoflac[®] Connect M fits vials of 20 mm rubber diameter
- Ecoflac® Connect L fits vials of 32 mm rubber diameter

4 Dilute

Reconstitute



Turn the system upside down. Squeeze and release the walls of Ecoflac[®] plus in order to inject fluid into the medication vial. Repeat until required volume is present in the vial.

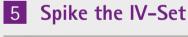

CLOSED SYSTEM



Turn the system so that the medication vial facing upwards. Apply pressure with two hands on Ecoflac[®] plus, to dilute the medication into the carrier solution. Label the container per institution SOP.





Remove the second aluminium foil from the Ecoflac[®] plus. Spike and prime the IV-Set (e. g. Intrafix[®] SafeSet).



To view a handling video for Ecoflac[®] Connect or find more information please visit

https://www.bbraun.com/en/products/b/ecoflac-connect.html

or scan the adjacent QR-code.

B. Braun Melsungen AG | Hospital Care | 34209 Melsungen | Germany | Tel. +49 5661 71-0 | www.bbraun.com

Date of last revision: 09.2018