

Intrafix® SafeSet Flush

IV-Administration Set for improved drug application

1 SPIKING THE DRUG CONTAINER AND PRIME THE IV-SET



1. Unwrap the IV-set and check whether the air vent cap is closed and the roller clamp is open.
2. Insert spike of Intrafix® SafeSet Flush into drug container.
3. Close roller clamp.
4. Hold the drip chamber upside down and open the roller clamp. Fill the drip chamber up to the green ring.
5. Turn drip chamber back into initial position and squeeze the drip chamber once in order to allow the system to fill. Prime the set completely with the drug. Close white snap clamp of Intrafix® SafeSet Flush.
6. Connect the IV-set to the cannula.

2 START APPLICATION



1. Open white snap clamp of Intrafix® SafeSet Flush and control flowrate by adjusting the roller clamp. Infuse the drug completely.
2. Disinfect needle-free connector of Intrafix® SafeSet Flush.

3 FLUSH THE IV-SET AND DISPOSAL



1. Connect suitable syringe (e. g. Omniflush®) to the needle-free connector of Intrafix® SafeSet Flush and flush the IV-set. Flushing with a suitable syringe (e. g. Omniflush®) helps to reduce the residual volume of the drug in the infusion line. White snap clamp should stay open and roller clamp in flow position or closed.



For more information about risk prevention in infusion therapy, please visit:

www.bbraunforsafety.com

Remark: Leave the roller clamp in flow position and allow the flushing solution to infuse at the same rate as the prescribed drug, or close it. This allows full delivery of the prescribed dose. **Roller clamp should not be completely open when flushing with a syringe! Avoid flushing with high pressure.**

Always double check the level of liquid in the drip chamber. If necessary, refill the drip chamber by squeezing.

The system can be disconnected from the cannula and disposed.