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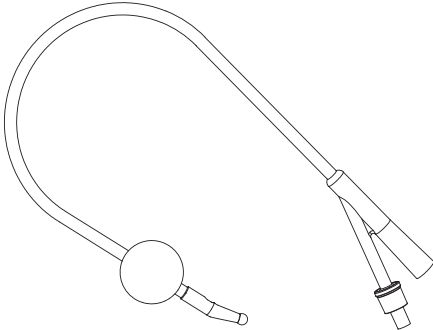
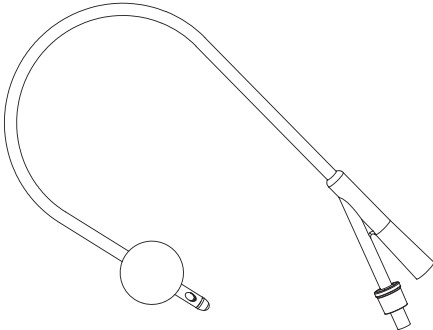
Urimed® Cath All Silicone Foley Catheter

DESCRIPTION

The Silicone Foley Catheters are available in 2-way and variant tips. They are made from silicone; consist of shaft, drainage funnel, inflation funnel balloon and valve.

The product is sterile and intended for single use only.

INDICATIONS

The Silicone Foley Catheters are indicated for bladder drainage of urine.

It is intended to be used by trained person or trained medical staff for drainage of the patient's bladder by inserting the catheter into the vesical cavity of the bladder through the urethra. It can be retained in the human body for up to 90 days.

CONTRAINDICATIONS

For transurethral catheterisation

- Acute urethritis
- Acute prostatitis
- Acute epididymitis

WARNINGS / PRECAUTIONS

- Do not over inflate the balloon. Refer to outer unit pack or funnel of catheter for balloon capacity.
- Inflate catheter balloon only with sterile water or sterile glycerine solution.
- The pre-filled syringe with sterile water or sterile aqueous glycerine solution (if included) is for inflation of the balloon only. Not for injection.
- The empty syringe (if included) is for deflation of the balloon before the removal of the catheter.
- The product is intended for use by physicians trained or suitably trained health care professional and aseptic technique must be practised.
- Do not use petroleum-based lubricants and silicone oil lubricants.
- Do not use if package has been opened or damaged.
- Do not clamp catheter shaft. It may damage catheter and prevent deflation.
- Do not use needle to inflate balloon. Use a Luer syringe (Luer slip or Luer lock).
- For single patient use only.
- Do not use after the expiry date.
- Patients should be regularly monitored as determined by a physician.
- Ensure local cleaning and hygiene protocols are followed to keep the catheter and meatus as clean as possible.
- If difficulty is encountered aspirating the balloon with a syringe, a rare and infrequently reported event, the leg of the catheter with the valve should be cut with a sharp scissor at the bifurcation or the balloon ruptured according to

established procedures reported in medical literature. Should it be necessary to rupture the balloon, care must be taken to remove all fragments from the patient's bladder.

- Incorrectly positioned catheters can cause urethral damage if the balloon is inflated contrary to instructions within the urethra.
- During long-term use, the balloon inflation volume should be regularly monitored in case any clinical signs of deflation occur, such as bypassing of urine or urethral pain. If necessary, the balloon volume should be adjusted.
- The balloon must be completely deflated before catheter removal.
- If deflation does not occur, established procedures should be followed and must be performed by a physician or suitably trained health care professional.
- This is a disposable product for single use, any sterilisation or reuse of the product can result in hazard to the patient.
- Precautions shall be taken when discarding device and disposal of the device shall be made in accordance with applicable national regulations for biological hazardous waste.

POTENTIAL COMPLICATIONS & ADVERSE REACTIONS

- Possible complications and adverse reactions known to be associated with indwelling foley catheters include irritation or inflammation (urethritis) of the urethral mucosa, blockage of the catheter due to encrustation, catheter-induced infections (as urinary tract infections), septicaemia, bladder spasm and abdominal discomfort.
- Long term use of catheter has the risk of urinary tract infection, sphincter relaxation and urethral relaxation.
- Patients should be routinely monitored in accordance with accepted procedures and the catheter shall be removed after a suitable interval as determined by a physician or other suitably qualified personnel.

DIRECTIONS FOR USE**※ For Transurethral Catheterisation**Preparation:

- The following additional items are required for catheterisation:
 - Sterile field, Sterile gloves, Items (sterile, based on established techniques) required for cleaning the patient meatus, Luer syringe with sterile water or sterile aqueous glycerine solution for balloon inflation (if not included), Empty syringe for balloon deflate (if not included), Sterile dressings
 - Urine drainage device

- Place male patients in supine position, female patients in lithotomy position.
- Wash and dry hands thoroughly.
- Using aseptic technique, remove the catheter from its pouch and place it on a sterile field. Put on sterile gloves and remove the catheter sleeve.
- Test the catheter for leakage or deflation of the balloon by inflating the balloon with the required volume (marked on the catheter funnel) of fluid prior to catheterisation. Release the syringe plunger and allow the balloon to deflate. Only use gentle aspiration, if necessary, to deflate the balloon.
- Lubricate the tip and the shaft of the catheter with a water-based lubricant.
- Clean the opening of the urethra and the surrounding area, using established techniques.

Catheterisation:

- With the uncontaminated hand, pick up the catheter from the sterile field.
- Carefully insert catheter tip into the bladder (normally indicated by urine flow), and then a further 3 cm to ensure balloon is also inside it.
- For catheters with Tiemann Tip, the tip should be facing pointed towards the ceiling or patient's face, assuming the patient is supine.
- Advance the catheter further to ensure that the balloon is located beyond the bladder neck, within the bladder.
- Inflate the catheter balloon with sterile water or sterile aqueous glycerine solution, at the discretion of the physician.
- Use the syringe or sterile glycerine solution to inflate the balloon with the required fluid volume (marked on the catheter funnel).
- Slowly retract the catheter until some resistance is felt to ensure that the balloon is correctly located within the bladder, before the bladder neck.
- Connect the catheter to a urine drainage device.
- Observe urine flow.

Catheter removal

- For removal of the catheter, deflate the balloon by inserting a Luer syringe into the inflation valve. Release the syringe plunger and allow the balloon to deflate. Only use gentle aspiration, if necessary, to deflate the balloon. The balloon should spontaneously deflate.
- Discard the catheter according to hospital protocol.

※ CATHETER CARE

- Encourage patients to drink more water to achieve the purpose of washing catheter.
- Found that urine turbidity, precipitation, crystallisation should be for bladder irrigation.

- The Patients should be aware of the protection of the catheter when bathing or rubbing, and should not immerse the catheter in water.
- If the catheter is accidentally removed or the confinement of the catheter is destroyed, the catheter should be replaced immediately.
- Always keep the urine drainage bag below the level of the bladder to allow urine to drain by gravity. Do not place the drainage bag on the floor.
- Ensure local cleaning and hygiene protocols are followed to keep the catheter and meatus as clean as possible.
- Conduct hand hygiene immediately before and after any manipulation of the catheter and the urine drainage system. Wear disposable gloves when handling the system.
- Maintain unobstructed urine flow.
- Ensure that the catheter and collection tube do not kink.
- Empty the urine collecting bag using a separate container for each patient, avoid splashing, and prevent contact of the drainage funnel with the non-sterile collection devices.
- The part of the catheter outside the patient and the catheter entry site (urinary meatus) must be kept as clean as possible and must be inspected at least daily. If pain (burning) occurs in the region of the entry site of if any other signs of inflammation such as fever are observed of if the catheter stops draining urine, a doctor should be informed immediately.
- This can happen especially with silicone balloons which are inflated with sterile water or sterile glycerine solution, because of their particular material characteristics. In order to prevent this from happening, the inflation liquid should be withdrawn from these catheters at least once a week. Then, immediately re-inflate the balloon with the stated nominal filling volume.

※ LONG-TERM USE CARE

- The patient should wash his hands before and after any catheter operation.
- The area around the catheter should be rinsed with a mild soap box at least twice a day.
- The patient taking 2 to 3 litres of water a day can help wash the catheter and keep the urine clear.
- Avoid constipation.
- The patient should regularly drain urine bags.
- The patient should always keep the drainage system closed.

CAUTION

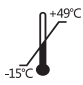
- For single use only.
- Sterile if package is unopened or undamaged.
- The size and balloon capacity is marked on outer unit pack and funnel of catheter.

- Any serious incident that may have occurred in relation to the device should be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

STORAGE CONDITIONS

- Store in a dark, cool and dry place.
- Avoid freezing and excessive heat.

MEANING OF SYMBOLS ON PACKAGE

	TEMPERATURE LIMIT		STERILISED USING ETHYLENE OXIDE		DO NOT USE IF PACKAGE IS DAMAGED
	KEEP DRY		KEEP AWAY FROM SUNLIGHT		DO NOT REUSE
	AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY		MANUFACTURER		USE BY
	DATE OF MANUFACTURE		BATCH CODE		CONSULT INSTRUCTIONS FOR USE
	MEDICAL DEVICE		WARNING		SINGLE STERILE BARRIER SYSTEM

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WELLEAD MEDICAL CO., LTD.
Shanghai International Holding Corp. GmbH (Europe)