Distribué par / Verdeeld door / Vertrieb / Distribuido por / Distributed by / Distribuito da / Gedistribueerd door /
Дистрибуция от / Dovăzi / Forhandles af / Maahantuoja / Forgalmazza / 販売元 / Dystrybutor / Distribuit de
/ Distribuira / Дистрибуьотер / Distribuerad av / Zastopnik / Distribútor da :

BE B. Braun Medical N.V./S.A.
1831 Diegem - Belgium

DE B. Braun Melsungen AG
34212 Melsungen - Germany

ES B. Braun Medical SA
08191 Rubí (Barcelona) - Spain

FR B. BRAUN MEDICAL SAS
92210 Saint-Cloud - France

GB B. Braun Medical Ltd.
Sheffield, S35 2PW - United Kingdom

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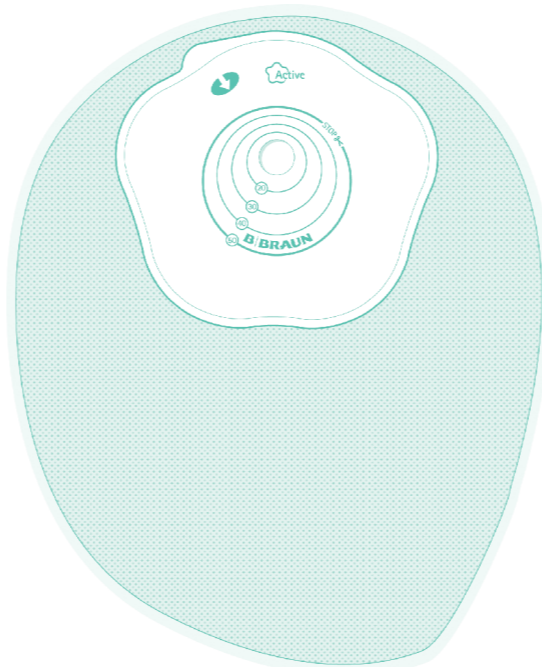
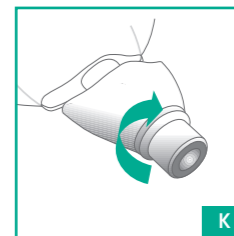
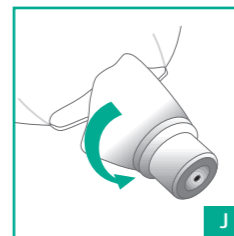
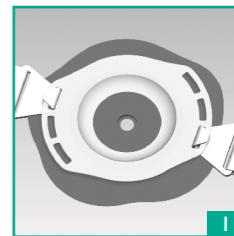
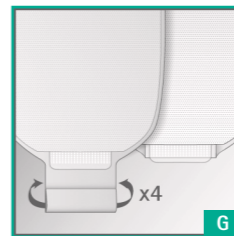
4 Use of drainable pouch Roll'Up:

Closing of the outlet:

- Fold the sleeve four times using the guide and fold the tab back.
- To seal the pouch, apply pressure on all the length of the two self-fastening strips facing each other.

Please note: the closed sleeve is hidden under the soft backing material (G).

Emptying the pouch:

- Position yourself to carry out the drainage, release the two self-fastening strips and unroll the sleeve. Be careful, the drainage will start at this point. To fully open the outlet place your fingers on each side and press slightly (E).
- Once the drainage is complete, expel, as much as possible, any matter still in the sleeve by pressing it lightly from top to bottom. Carefully clean its extremity (be careful: use non-shedding material) (F).
- Then close the pouch.

To close the pouch, fold the sleeve as indicated above.

5 Removing your appliance:

Pull slightly on the top of the pouch and remove the pouch from the top to the bottom while maintaining gentle pressure on the skin around the stoma with your other hand.

6 Use of the patch cover for filter protection:

If you wish to inactivate the filter before you take a shower or a bath, stick an adhesive patch on the top of the pouch

so that the hole for gas evacuation is blocked (D).

7 Urostomy pouches

Handling the drainage tap:

- Point the drainage system towards the toilet bowl, slowly turn the cap counter-clockwise until the desired flow rate is obtained (J).
- To close the cap once the pouch has been emptied, turn clockwise until it has returned to its original position (K).

Connecting to an overnight or leg pouch:

- Connect the pouch cap to the overnight or leg pouch using the supplied connector.
- Open the tap on the emptiable pouch to allow the urine to flow. Close the tap before disconnecting from the collection pouch.

**Availability may vary from one country to another.*

National regulations for the destruction of potentially infectious materials must be observed.

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