# Infusomat<sup>®</sup> Space Line

Type Enteral Nutrition

Type Enteral Nutrition ENFit

---

<b>GB</b>	Instructions for use
<b>DE</b>	Gebrauchsanweisung
<b>BG</b>	Инструкция за употреба
<b>BR</b>	Instruções de utilização
<b>CN</b>	使用说明
<b>CZ</b>	Návod k použití
<b>DK</b>	Brugsanvisning
<b>EE</b>	Kasutusjuhised
<b>ES</b>	Instrucciones de uso
<b>FI</b>	Käyttöohje
<b>FR</b>	Mode d'emploi
<b>GR</b>	Οδηγίες χρήσης
<b>HU</b>	Használati utasítás
<b>IT</b>	Istruzioni per l'uso
<b>LT</b>	Naudojimo instrukcija
<b>LV</b>	Lietošanas instrukcija
<b>NL</b>	Gebruikersinformatie
<b>NO</b>	Bruksanvisning
<b>PL</b>	Instrukcja użytkowania
<b>PT</b>	Instruções de utilização
<b>RO</b>	Instrucțiuni de utilizare
<b>RS</b>	Uputstvo za upotrebu
<b>RU</b>	Инструкция по применению
<b>SE</b>	Bruksanvisning
<b>SI</b>	Navodila za uporabo
<b>SK</b>	Návod na použitie
<b>TH</b>	คำแนะนำในการใช้งาน
<b>TR</b>	Kullanım Kılavuzu

---

**B | BRAUN**

B. Braun Melsungen AG  
34209 Melsungen  
Germany

[www.bbraun.com](http://www.bbraun.com)

**GB** Administration set for enteral nutrition only. For application and admixture of enteral nutrition by compatible pumps or gravity. **Indication:** Compatible to enteral fluids and to mother milk.

**Contraindications:** Not for administration or admixture of liquids and drugs, which are meant to be infused via an IV-access e.g. I.V. solutions, blood and blood products, light sensitive drugs, parenteral nutrition solutions.

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


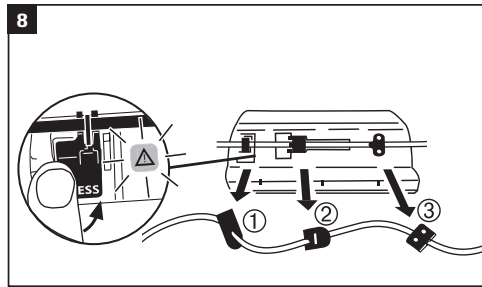
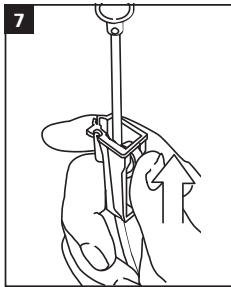
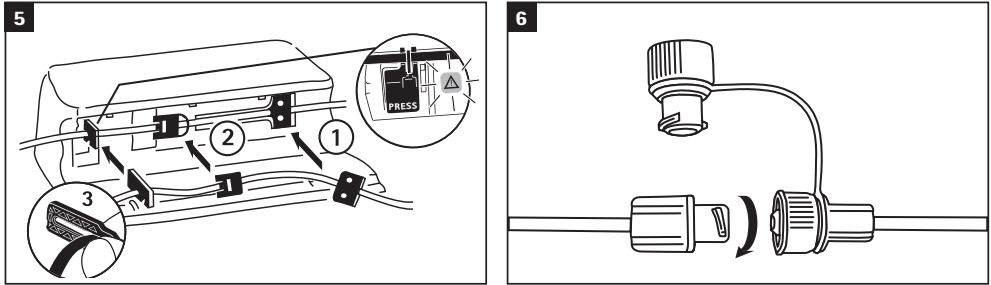
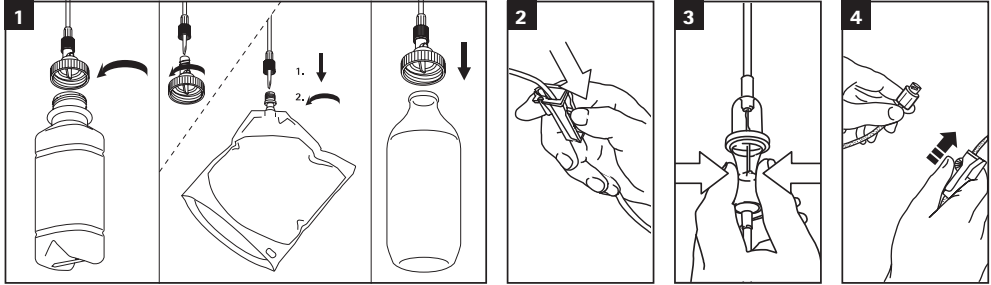
**P** = Pressure (Druck)
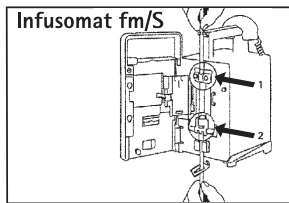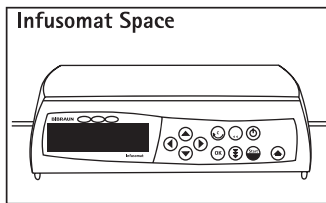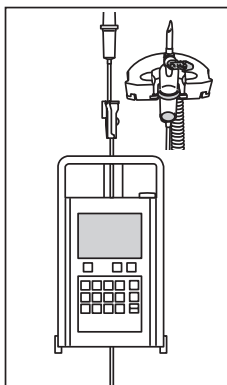
## Warning notices

**GB** Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.

Sterile, non-pyrogenic. Do not use if protective caps are loose or missing or if packaging is damaged. Caution! If a pneumatic cuff is used during infusion, remove air from the container beforehand. Observe general safety precautions!

Please observe Instructions for Use of B. Braun Infusion pumps and parallel infusions. When using Infusomat Space without a drip sensor, use a volume setting smaller/ equal the filling volume of the Enterofix/ nutrition bag.

If Infusomat Space Line with NEUTRAPUR tubing is used please select respective „Space Line NEUTRAPUR“ setting.

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