- EN** Instructions for use
- DE** Gebrauchsanweisung
- ES** Instrucciones de uso
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- BE** Инструкции по применению
- BG** Инструкции за употреба
- CS** Návod k použití
- DA** Brugsanvisning
- EL** Οδηγίες χρήσης
- EE** Kasutusjuhend
- FI** Käyttöohjeet
- HR** Upute za upotrebu
- HU** Használati utasítás
- JA** 使用方法
- LV** Lietošanas instrukcija
- LT** Naudojimo instrukcijos
- NO** Bruksanvisning
- PL** Instrukcja obsługi
- PT** Instruções de utilização
- RO** Instrucțiuni de utilizare
- RU** Инструкция по применению
- SK** Pokyny na použitie
- SL** Navodila za uporabo
- SR** Uputstvo za upotrebu
- SV** Bruksanvisning
- TR** Kullanım Kilavuzu

Puncture set, exchange set or dilation exchange set for suprapubic percutaneous catheter insertion.

Contents

Cystofix®/Cystofix® Paed

Drainage catheter preassembled in its puncture cannula, protective tubing (for Cystofix® Puncture Set), stopper for balloon catheter only (doesn't apply for Cystofix® Paed), drainage bags (2L) with drain tap and protective cap (for the Cystofix® Puncture Set with bags).

Cystofix® Minipaed

Soft, coated mandril wire mounted in the drainage catheter, adapter, puncture needle, splittable introducer with safety sheath, guide wire (for Cystofix® Minipaed Exchange sets).

Indications

Catheterisation of the bladder after surgery, in case of bladder dysfunction, urinary retention, for diagnostic purposes, for urine assessment.

Contraindications

Carcinoma of the bladder, anticoagulation and antiplatelet treatment, abdominal wall sepsis, presence of a subcutaneous vascular bypass in the suprapubic region.

Side effects

In case where, despite the special design and material of the catheter, a deposition of urinary components on the catheter material occurs, consider pH of urine, concentration of urine and/or possible urinary infection. Occasionally, strangury due to foreign body irritation may occur.

Recommendations

- **Mandatory:** the insertion of a suprapubic percutaneous catheter and management of the patient with the SPC must be undertaken only by staff who has acquired appropriate medical competencies.
- Please take into account current guidelines and recommendations.
- Cystofix® products are not intended to be used for long term implantation.
- Cystofix® polyurethane catheter is indicated to be used for short term catheterisation.
- Cystofix® silicone balloon catheter is indicated to be used for long term suprapubic catheterisation.
- Do not introduce the catheter beyond the last catheter mark.

Re-use of single-use devices creates a risk for the patient or user. The device can be contaminated and/or its functional capacity impaired. Contamination and/or limited functionality of the device may lead to injury, sepsis or death of the patient.

Urine bags contain DEHP. Animal experiments have shown DEHP to be potentially toxic to reproduction. Given the current state of scientific knowledge, a risk (in case of long-term use) especially for premature male babies cannot be completely excluded. As a precaution, the use of medical products containing DEHP should be restricted to short-term use for pregnant women, breastfeeding mothers, infants, and children.

- Do not puncture in cases of: insufficiently filled bladder.
- Do not puncture at distance of the median line of the body : risk of bleeding!
- Do not withdraw the catheter from the cannula if the catheter tip protrudes beyond the cannula tip (without the protective sheath in place), catheter tip may be cut off!
- Separate cannula halves cautiously. Edges of splitted cannula are sharp and may cut if handled inappropriately!
- When exchanging the catheter, do not insert the guidewire with the stiff tip first.
- Check the catheter function regularly; the physician will decide how often and which method should be used for this check.
- Do not use if the packaging is damaged.
- For the balloon catheters : do not inflate more than indicated on the funnel.

Other indications: None

Interactions: None

Application

First puncture

Cystofix® Puncture Set, Cystofix® Paed

- Open the blister pack then prepare the set component on a sterile area.
- Then, bladder must be filled according current guidelines and recommendations and it must be possible to localize it.
- Evaluate whether local anaesthesia is advisable according current guidelines, then perform it all along the puncture track until the

bladder, then test the introduction into the bladder.

- Perform a slight incision on the superficial skin at the origin point of the local anaesthesia if applicable.
- Connect the catheter on the urine drainage bag.
- Keep the protective cannula sheath in place during the catheter preparation stage.
- Gently pull out the catheter inside the cannula until the last mark.
- Remove the protective sheath.
- Puncture on the median line of the body above the symphysis (2-3 cm) vertically to the abdominal wall.
- After reaching the bladder urine appears in the catheter tube.
- Push the catheter inside the bladder until the double mark, then do not pull it out anymore.
- Withdraw the cannula out of the puncture track until the tip of the cannula bevel.
- Split the cannula by bending the two wings.
- Pull apart and remove the halves of the cannula.
- Fix the catheter on the skin to prevent its removal during the application time.
- Fix the balloon catheter on the skin to prevent the pulling strength on it.

Exchange of catheter

- Fill the bladder at approx. 50%.
- Insert the guide-wire into the catheter – flexible tip first.
- Remove the catheter.
- Insert a new catheter by sliding on the guide wire

Exchange of catheter with dilator

- Procedure as above, but after removal of the old catheter dilate the puncture track by means of a dilator/sheath-unit.
- Remove dilator out of the sheath, insert the catheter into the sheath.
- Withdraw sheath on catheter out of the puncture track and remove it by splitting.
- Fix the catheter on the skin.

When using a balloon catheter

- Pull out the catheter with balloon until the tip does not appear anymore in the cannula bevel.
- Fill up the balloon with the specified volume by using a sterile mixture of glycerine and water in proportion 1 to 9. This prevents evacuation of the balloon.
- Withdraw catheter up to the point where the balloon makes contact with the bladder wall.
- If needed, in case of high pulling strengths, fix the balloon catheter on the skin.

When using the Cystofix® Minipaed

- Remove the puncture needle and its splittable cannula together from the protective sheath.
- The puncture is done with the puncture needle together with a splittable cannula.
- Check that the urine comes out of the end of the puncture needle by opening the protective cap.
- Then remove the puncture needle from the splittable cannula in place.
- Push totally the soft mandril into the catheter tube and lock the mandril-cap on the male luer-lock connector.
- Now the catheter is straight and ready to be inserted.
- Insert the catheter and its soft mandril together into the splittable cannula in place until the second red mark.
- Unscrew the soft mandril cap from the catheter connector and remove it from the catheter in such a way that the pigtail is reshaped inside the bladder.
- Withdraw the splittable cannula out of the puncture track and move it along the catheter until the cannula tip appears.
- Split the cannula in two parts then pull them apart and remove the halves of the cannula.
- Fix the catheter on the skin appropriately to prevent its removal during use.

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B. Braun Medical SAS