



# SUPERFILLER

PROTECTIVE SKIN BARRIER PASTE



# SUPERFILLER PROTECTIVE SKIN BARRIER PASTE



## HOW TO APPLY:

- 1 Wash your hands and the skin around the stoma.
- **2** Dry both hands and the skin around the stoma carefully.
- 3 Apply some paste around the stoma or directly on the skin protector/ wafer.
- <sup>4</sup> Then apply the skin protector /wafer.





#### PRODUCT INFORMATION:

Product Description	Size	Product Code
Unit Tube	60g	F05008

## ADVANTAGES:

- Ideal to fill in folds around the uneven peristomal skin surfaces
- Easy to use and spread
- Prevents leakages and irritation

#### INDICATION:

- Protective skin barrier paste
- Colostomies Ileostomies Urostomies Fistulae
- For uneven peristomal skin surfaces
- Used when leakages occur because skin protectors do not adhere closely to patient's stoma.

#### **COMPOSITION:**

- Contains penpure 70
- Alcohol

Manufactured by: Avery Dennison Medical Ltd. | IDA Business Park | Ballinalee Road | Longford | N39 DX 73 | Ireland.

B. Braun Medical SAS | 26 rue Armengaud | 92210 Saint-Cloud - France Tél. 01 41 10 53 00 | Fax 01 41 10 53 99 | www.bbraun.fr | www.bbraun.com Societé par Actions Simplifiée with a share capital of 31 000 000 € | RCS Nanterre 562050856

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun. Non binding documents and photographs.